



DermaFoam™

Waterproof Foam Wound Dressing

DermaFoam is a highly absorbent foam dressing that wicks wound exudate to reduce the risk for maceration in moderate to heavily draining wounds. It has a waterproof polyurethane film backing to prevent strikethrough of exudate. DermaFoam is non-adhesive so it is easy to remove; it can be cut to size for irregularly shaped wounds and needs to be secured in place. DermaFoam Tracheostomy also available, featuring a keyhole opening for easy application around tracheostomy sites.

Features	Benefits
Highly absorbent foam dressing	<ul style="list-style-type: none"> • Supports moist wound healing and autolytic debridement • Minimizes risk of maceration • Absorption, support and cushioning help manage hypergranulation tissue
Waterproof polyurethane film backing	<ul style="list-style-type: none"> • Protects wound from external contamination • Prevents strikethrough of exudate
Can be safely worn for up to 3 days, based on wound condition and exudate level	<ul style="list-style-type: none"> • Reduces caregiver time and associated costs
Provides thermal insulation and protection	<ul style="list-style-type: none"> • Maintains an optimal healing environment
Can be cut to fit wound site	<ul style="list-style-type: none"> • Ease of use and improved patient comfort
Wide variety of sizes	<ul style="list-style-type: none"> • Convenient and comfortable

DermaFoam™

WATERPROOF FOAM WOUND DRESSING

INDICATIONS

A primary dressing that may be used for the management of acute or chronic partial and full thickness wounds/ulcers with moderate to heavy exudate.

INSTRUCTIONS FOR USE

- STEP 1** Prepare wound site and surrounding skin per protocol.
- STEP 2** Dressing may be cut to size prior to application.
- STEP 3** Apply foam surface directly onto wound.
- STEP 4** Secure with secondary dressing or tape.
- STEP 5** Reapply as needed based on condition of wound, surrounding skin & dressing, or as directed by a qualified health professional.

GENERAL TIPS FOR USE

- Foam dressings are designed to be highly absorbent, while still being gentle to the skin, with minimal skin stripping upon removal.
- For use with moderate to heavy exudate.
- DermaFoam must be secured with a secondary dressing.
- Ensure that the waterproof backing faces away from the wound bed when applying.

DermaFoam™

WATERPROOF FOAM WOUND DRESSING

DermaFoam



Product	Size	Reorder #	Units
DermaFoam	2x2	00290E	10/Box
DermaFoam	4x4.25	00291E	10/Box
DermaFoam	6x6	00292E	10/Box
DermaFoam	Heel/Elbow	00293E	5/Box

DermaFoam Tracheostomy



Product	Size	Reorder #	Units
DermaFoam Tracheostomy	3.5x3.5	45330	10/Box

DermaRite®