PREGNANCY TEST

CLIA '88

WAIVED

WHEN YOU HAVE TO BE RIGHT ... YOU NEED TO HAVE THE BEST

QuPID[®] Urine Specimen Only

5

5

QuPID[®] Plus E.R.

Sample

S

QUPID® Plus

STANBIO

ABOBATORY

Sample

С

QuPID®

S

Sample

Serum or Urine Specimen

Detects 10 mIU/mL hCG or greater in serum.

QuPID[®] Plus E.R.

Detects 20 mIU/mL hCG or greater in urine specimens.

QuPID[®] Plus Serum or Urine Specimen

Detects 20 mIU/mL hCG or greater in both urine and serum specimens.

QuPID,[®] QuPID[®] Plus & QuPID[®] Plus E.R.


The Visibly Better One-Step Pregnancy Test

The World's Source For Clinical Diagnostics

PREGNANCY TEST

EVERYTHING YOU NEED IN A CONVENIENT POUCH

Everything you need is included in one foil pouch (including dropper). The device features a clear, easy-to-read background and easy-to-use instructions right on the pouch to provide you with the highest level of reliability and usability.

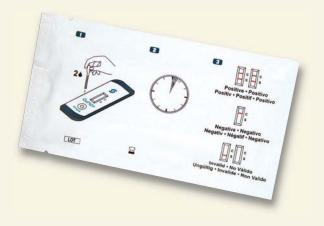

QuPID®, QuPID® Plus and QuPID® Plus E.R. Benefits

Exceptional Readability — Quick clearing background ensures easy-to-read results even with low-level specimens.

Fast — Positive results may be seen in as little as 10 seconds.

Sensitivity — Detects 20 mlU/mL hCG or greater in both urine and serum specimens. QuPID[®] Plus E.R. detects 10 mlU/mL hCG in serum.

Size

50 Test

2 x 5 mL

3 x 5 mL

25 Test / 50 Test

25 Test / 50 Test

Accuracy — >99%


Specificity —>99%

Products

QuPID® hCG Pregnancy Test Set QuPID® Plus hCG Pregnancy Test Set QuPID® Plus E.R. hCG Pregnancy Test Set hCG Bi-Level Urine Controls hCG Tri-Level Serum Controls

General Test Procedure

- 1. Remove QuPID[®] device and disposable dropper from foil pouch.
- 2. Label the test device with patient or control identification.
- 3. Dispense two drops of the specimen into the round sample well.
- 4. Read result as positive or negative.

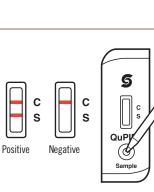

Reference No.

1250-050

1225-205

1235-305

1220-025 / 1220-050

1230-025 / 1230-050





DISTRIBUTED BY

S	1	Т	Α		N		В				
	L	A	в		R	A	т		R	Y	

The World's Source for Clinical Diagnostics

- General & Special Chemistry
- Pregnancy
- Point-of-Care
- Rapid Tests
- Therapeutic Drug Monitoring
- UrinalysisClinical Instrumentation

Occult Blood

Serology

Pipettors

1261 North Main Street Boerne, Texas 78006 USA Toll Free: 866.STANBIO (782.6246) Tel: 830.249.0772 Fax: 830.249.0851

stanbio@stanbio.com www.stanbio.com



Lit # RSS0001.01 ©2007 Stanbio Laboratory Printed in USA.