



## SafeWash™

### Sterile Saline Wound Wash

SafeWash is a sterile saline wound wash spray containing 0.9% sodium chloride. Delivers safe irrigation pressure to remove bacteria and debris painlessly. Revolutionary point-and-spray technology sprays even upside down. Preservative free and drug free. Contains no CFC's.

Features	Benefits
Point and spray technology	<ul style="list-style-type: none"><li>• Helps dislodge bacteria and debris</li><li>• Can be used upside down for hard to reach places</li></ul>
Non-stinging	<ul style="list-style-type: none"><li>• Gentle and painless</li></ul>
Preservative and drug free	

# SafeWash™ Saline

## STERILE SALINE WOUND WASH

### INDICATIONS

- To cleanse wounds, scrapes, and minor burns.

### INSTRUCTIONS FOR USE

Before use, expel a short stream of SafeWash™ Saline to clear nozzle.

STEP 1 To dispense, simply aim nozzle and press.

STEP 2 Flush affected area with SafeWash™ Saline. Adjust distance as necessary.

STEP 3 Reapply as often as needed or as directed by a qualified health professional.

Product	Size	Reorder #	Units
SafeWash Saline	7.1 OZ	00245	12/Case