

Derma**Klenz**™

Wound Cleanser

DermaKlenz is a non-irritating, pH balanced wound cleanser designed to facilitate the removal of organic material, slough, blood clots, other wound debris and dead tissue, without disrupting the wound bed. Formulated with zinc to promote healing.

Features	Benefits	
Formulated with Zinc and Vitamin B6	 Key ingredients for wound bed preparation to support healing 	
No detergents added	No rinsing required	
Adjustable spray nozzle	 creates direct stream or gentle mist with appropriate PSI irrigation 	



DermaKlenz[™]

WOUND CLEANSER

INDICATIONS

Use to cleanse wound and prepare wound bed for primary dressing.

INSTRUCTIONS FOR USE

STEP 1 Hold bottle about 6 inches from affected area.

STEP 2 Adjust nozzle for fine mist or irrigating stream

STEP 3 Flush wound and thoroughly and wipe gently with clean gauze pad.

STEP 4 Repeat as necessary.

GENERAL TIPS FOR USE

• No need for additional irrigation set.



Derma**Klenz****

WOUND CLEANSER



Product	Size	Reorder #	Units
DermaKlenz	4 0Z	00243	12/Case
DermaKlenz	8 OZ	00249	24/Case

