

REF 6216

UNITS - 1

RX Only



Not Made
With Natural
Rubber Latex



Not Made
With DEHP



FILTER

Bacterial/Viral Filter

22mm OD/15mm ID x 22mm ID 99.99%

Filtration Efficiency 0.83cm H₂O@ 30LPM

Resistance 41.85ml Dead space

SUGGESTED PROCEDURE:

1. Filters may be fitted at patient end or machine end of circuit.
2. HMEs and HME Filters must be fitted at patient end of circuit.
3. Do not use between the patient and a nebulizer or humidifier.
4. Change the filter as required or as per protocol.

CAUTION: For single patient use only. Use of sterilization or cleaning solutions may leave this device nonfunctional. Store in a cool dry place.

NOTE: The presence of water droplets and/or salt crystals observed in the HME indicates normal function.

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com

EC

REP

MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany
Assembled in MEXICO
LABEL PN 78103, Rev. 03



LOT