

EZ-Cleans Plus® Kit Directions

Kit Number: (17121)

Kit Contents:

- Pair of Vinyl Gloves
- Red Z® Solidifier (10g. Pouch)
- Scoop/Scraper
- SaniZide Pro® Germicidal Wipe (Dimensions: 8" x 8")
- Red Bio-hazard Waste Bag (Dimensions: 24" x 24")
- Twist Tie
- p.a.w.s.® Antimicrobial Hand Wipe (Dimensions: 5" x 8")
- Paper Towel
- Instructions

Directions:

1. Put on disposable gloves.
2. Sprinkle Red Z® over spilled area. Allow liquid to congeal for safer handling and transport.
3. Remove gelled material with scoop and scraper. Carefully place in red bio-hazard bag.
4. Use SaniZide Pro® Disinfectant Wipe to clean contaminated surface.
5. Place all contaminated materials (including wipe and gloves) in the red bio-hazard bag. Seal bag with supplied twist tie. Dispose of bag as required by local, state and federal regulations for infected solid waste.
6. Wipe hands with p.a.w.s.® Antimicrobial Hand Wipe. Wash with soap and water as soon as possible.