

# Custom Brochure

CREATED

Grove Medical

This publication was constructed for you using the Brochure Builder application  
available at the Smiths Medical website. Visit us to create your own.

[www.smiths-medical.com](http://www.smiths-medical.com)

## EzPAP® Positive Airway Pressure System

PORTEX®

The EZPAP® positive airway pressure system is used to expand the lungs by increasing Functional Residue Capacity (FRC). Increasing FRC has been found to contribute to the prevention and reversal of atelectasis.

EzPAP® is the easy option for the prevention and treatment of atelectasis and a medical need for lung expansion therapy.

When incentive spirometry alone won't open patient's airways, expand your options with EzPAP®. It makes providing positive airway pressure positively easy. Simply connect to a flow meter (wall air or O2 for enhanced FiO2), adjust 5-15 lpm, and instruct the patient to breathe normally through the mouthpiece or mask. Just a few minutes of therapy, as needed - not for hours at a time.

EzPAP® features a pressure port for connection to a gauge (recommended for initial use with each patient), and a standard 22 mm OD fitting to accommodate a mouthpiece or 3 mask options.

The EzPAP® Positive Airway Pressure System includes the EzPAP® lung expansion device, mouthpiece, pressure port cap, and 7 feet of tubing. Three mask options are also available.

- Use with mouthpiece or mask attachment
- 22 OD patient end
- Scalloped ambient air inlet
- Pressure port with cap
- Complete procedural kit
- Disposable manometer
- Single patient use



## PRODUCT BENEFITS

This product benefits the patient by:

- Can be used with mask or mouthpiece attachment
- Allows use with inline nebulizer when fitted with tee adapter Prevents inadvertent occlusion
- Connects to gauge tubing
- Eliminates collecting components
- Eliminates the need to carry a manometer

## PRODUCT SPECIFICATIONS

### EzPAP® Positive Airway Pressure System

Reorder Code	Description	Units per Case
--------------	-------------	----------------

23-0747	EzPAP® System w/Mouthpiece.	10
---------	-----------------------------	----

23-0757	EzPAP® System w/Disposable Manometer	10
---------	--------------------------------------	----

23-1747	EzPAP® System w/pediatric mask	1
---------	--------------------------------	---

23-2747	EzPAP® System w/medium mask	1
---------	-----------------------------	---

23-3747	EzPAP® System w/large mask	1
---------	----------------------------	---

Notes: Each system includes EzPAP®, one 7 ft. oxygen tube and one pressure port cap.

### EzPAP® Positive Airway Pressure System (Not available in the EU)

EzPAP® Kit - Contains one pressure manometer (with gauge protector), three EzPAP® units (with mouthpiece), three Gauge Guards and ten 22-mm ID adaptors in a durable plastic box

**Reorder Code Units per Case**

23-6000 1