

Bard* Button Device Continuous Feeding Tube with 90° Adaptor

Sonde à gavage permanent, à utiliser avec le dispositif Button **Bard***
et avec Adaptateur à 90°

Dauerernährungssonde mit 90°-Adapter für **Bard*** Button-Instrumente

Sonda per alimentazione continua per dispositivo **Bard*** Button con adattatore a 90°

Tubo de Alimentación Continua con Adaptador de 90° para el Dispositivo Button **Bard***

Bard* Continuvoedingsbuis met haakse adaptor (90°) voor Button instrument

Sonda de Alimentação Contínua do Dispositivo Button **Bard*** com Adaptador de 90°

Σωλήνας Συνεχούς Σίτισης Συσκευής **Bard*** Button με Προσαρμογέα 90°

Bard* Button anordning med sonde til vedvarende ernæring med 90° mellemstykke

Bard* Button Instrument Kontinuerlig Matarslang med 90° Adapter

Bard* Button Device jatkuvan ruokinnan letku 90°:n adaptorilla

Bard* Button-enhet med kontinuerlig materør med 90° adapter

Bard* Button Eszköz - Folyamatos Tápláló Tubus 90° Adapterrel

Kontinuální výživová sonda zařízení **Bard*** Button s 90° adaptérem

Rurka do podawania pożywienia metodą wlewu ciągłego dla zestawu Button i łącznikiem pod kątem 90° firmy **Bard***

Bard* 90°Adaptörlü Button Cihazı Sürekli Besleme Tüpü

Трубка для непрерывного питания к низкопрофильному гастростомическому устройству производства компании **Bard*** с изогнутым на 90° адаптером

Information for Use

Informations pour l'usage

Gebrauchsinformationen

Informazioni per l'uso

Información para su uso

Informatie voor gebruik

Informações de Utilização

Πληροφορίες Χρήσης

Brugsanvisning

Bruksanvisning

Käyttöohjeet

Informasjon om bruk

Felhasználási tájékoztató

Informace pro použití

Instrukcja obsługi

Kullanım Bilgileri

Информация об использовании

BARD

Manufacturer



Bard Access Systems, Inc.

605 North 5600 West

Salt Lake City, Utah 84116 USA



1-800-545-0890 (U.S.A.)

1-801-522-5000

www.bardaccess.com

Technical or Clinical Support

1-866-893-2691 (U.S.A.)

medical.services@crbard.com

0732752 1205R Revised 5/2012

EC REP

Bard Limited

Forest House, Brighton Road

Crawley, West Sussex

RH11 9BP UK

Rx Only - Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

This product and packaging do not contain natural rubber latex



Attention, see instructions for use



NON-STERILE

Device Description

The **Bard*** Button Device Continuous Feeding Tube is a detachable 24" (61 cm) continuous feeding tube with 90° adaptor, including a clamp and 90° adaptor designed specifically for use with the **Bard*** Button replacement gastrostomy device.

The following chart indicates the appropriate continuous feeding tube to be used with the various sized Bard Button replacement gastrostomy devices:

CONTINUOUS FEEDING TUBE AND SIZE		BARD* BUTTON DEVICE AND SIZE	
REF 000256	18F (6.0 mm)	REF 000292	18F (6.0 mm) 1.2 cm
		REF 000282	18F (6.0 mm) 1.7 cm
		REF 000283	18F (6.0 mm) 2.4 cm
		REF 000284	18F (6.0 mm) 3.4 cm
REF 000258	24F (8.0 mm)	REF 000293	24F (8.0 mm) 1.2 cm
		REF 000285	24F (8.0 mm) 1.7 cm
		REF 000286	24F (8.0 mm) 2.4 cm
		REF 000287	24F (8.0 mm) 3.4 cm
		REF 000296	24F (8.0 mm) 4.4 cm
REF 000268	28F (9.3 mm)	REF 000261	28F (9.3 mm) 1.5 cm
		REF 000262	28F (9.3 mm) 2.7 cm
		REF 000263	28F (9.3 mm) 4.3 cm

Indications for Use

To provide enteral feeding in patients requiring long-term nutritional support.

Contraindications

Include use of this device with any product other than the **Bard*** Button replacement gastrostomy device; or for indications other than delivering feeding nutrients.

Warning

Intended for Single Patient Use. Use of this device in more than a single patient increases the risk of cross-contamination of infectious diseases between patients.

After use, product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.



Precaution

Too much pressure may cause the entire Button device to be pushed into the stomach or to be prematurely pulled out of the stomach.

Adverse Reactions

May include: minor wound infection or pressure necrosis at stoma site; leakage of gastric contents; gastrocolic fistula; and gastric separation leading to peritonitis, sepsis and death. All of these potential complications increase in likelihood with improper placement. If the stoma measuring device is pulled too tightly against the stomach wall an improperly sized, i.e., too short, replacement device may be selected.

Instructions for Use

1. Inspect package contents for damage. If damaged, do not use.
 2. To administer feedings:
 - If a pump is used, verify proper pump and set connections. Follow manufacturer's instructions to set the pump at the prescribed feeding rate.
 - Prime feeding set or fill feeding syringe, attach to continuous feeding tube, then prime the tube if desired.
 - Open the safety cap on the Button device and insert 90° adaptor completely, using a slight twisting action and slight pushing pressure. The adaptor should be completely inserted to ensure a secure fit.
 - The device is now ready for use. Once started, feeding will flow through the continuous feeding tube into the Button device.
 3. Once feeding is complete, flush the continuous feeding tube and the **Bard*** Button replacement gastrostomy device with an amount of water appropriate for the patient.
 4. Remove the continuous feeding tube with a slight twisting action and slight pulling pressure.
 5. Close the safety cap to keep the lumen clean, the profile cosmetically pleasing and to minimize gastric reflux.
 6. Mechanically clean the continuous feeding tube with soap and warm water and thoroughly rinse.
-

WARNING: After use, product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.



An issued or revision date and a revision number for these instructions are included for the user's information on the first page directly beneath the telephone number of Bard Access Systems. In the event that two years have elapsed between this date and product use, the user should contact Bard Access Systems to see if additional product information is available. (Telephone Number: 1-800-545-0890 in the USA, or 801-522-5000)

www.bardaccess.com

Technical or Clinical Support 1-866-893-2691 (USA).
medical.services@crbard.com

*Bard is a trademark and/or registered trademark of C. R. Bard, Inc.

© 2012 C. R. Bard, Inc. All rights reserved.

Rx Only : La loi fédérale (E.-U.) n'autorise la délivrance de ce système qu'à un médecin ou sur prescription d'un médecin.

Ce produit et son emballage ne contiennent aucune trace de latex en caoutchouc naturel



Attention, voir le mode d'emploi



NON-STERILE

Description du dispositif

La sonde à gavage permanent pour dispositif Button **Bard*** est une sonde détachable de 61 cm (24") de longueur, munie d'un clamp et d'un adaptateur à 90° qui ont été tout spécialement conçus pour être utilisés avec le dispositif de remplacement Button **Bard*** pour gastrostomie.
