



Short Instructions


The **VACUETTE® QUICKSHIELD** Safety Tube Holder,
together with the **VACUETTE®** Blood Collection Tubes and needles,
is suitable as a safe single-use system for everyday venous blood collection.

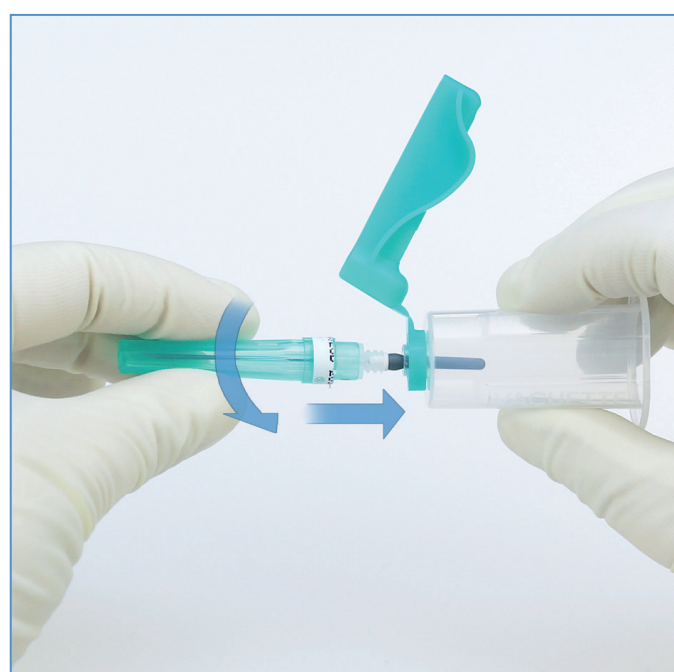

1 Thread the needle

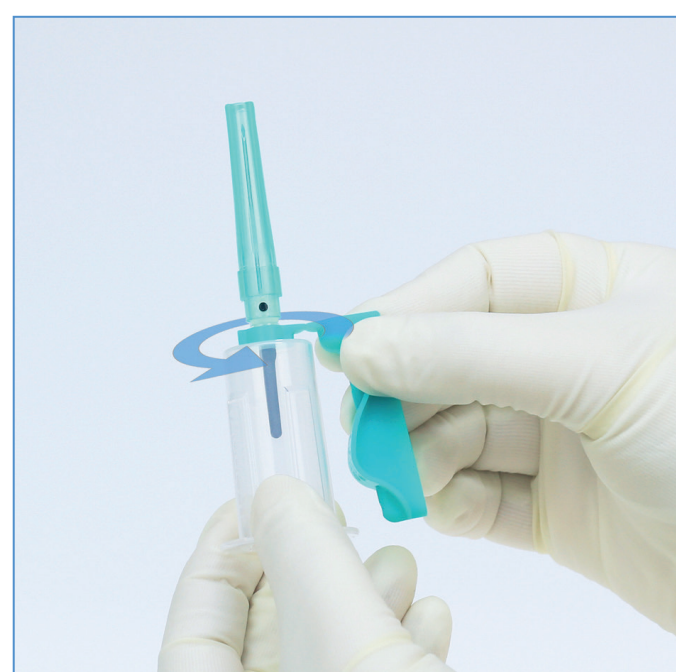

2 If desired, re-position
safety shield

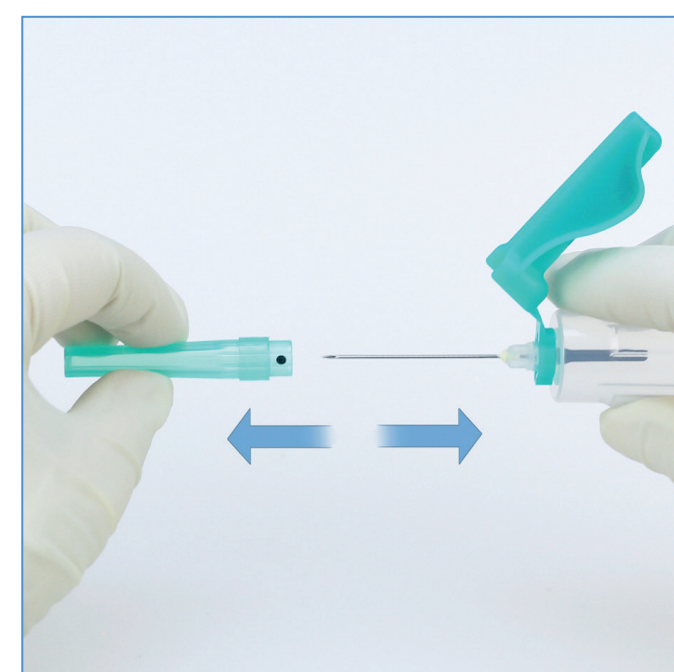

3 Remove protective
needle cap




4 Venipuncture



5 Insert the tube



6 Remove the tube

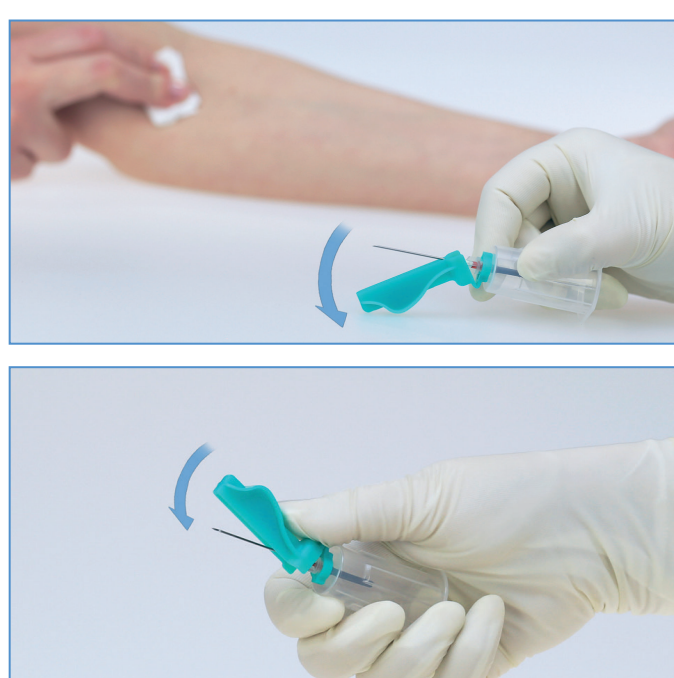

7 Activate the safety
mechanism
(with the aid of solid
surface or thumb)



8 Audible click
indicates activation



9 Disposal

* For the complete
handling recommendation
please refer to the
Instructions for Use for each product.