



Puracyn[®] Plus Professional Formula Antimicrobial Hydrogel is a next-generation wound care solution designed to add and maintain moisture in the wound bed and facilitate autolytic debridement. The Antimicrobial Hydrogel has an expansive safety profile that won't harm healthy tissues and works in concert with suite of associated products that deliver a synergistic approach to wound bed preparation and management.

EXPANDING THE POSSIBILITIES OF WOUND CARE:

Puracyn Plus Professional Formula Antimicrobial Hydrogel simplifies the healing process through the use of an advanced hypochlorous technology with moisturizing properties. Hydrogel is intended for use on wounds that have little or no exudate.

PURACYN[®] PLUS TECHNOLOGY:

The Antimicrobial Hydrogel is uniquely formulated by combining electrochemically produced hypochlorous solution with a nanostructured viscosity enhancer and pharmaceutical grade phosphate blends. The resulting formulation is clear, pH-balanced hydrogel optimal for a wide range of wound and skin conditions.

PURACYN[®] PLUS STABILITY:

The Antimicrobial Hydrogel has a two-year shelf life. This stability is a result of the unique solution chemistry based on a proprietary manufacturing process.

PURACYN[®] PLUS BENEFITS:

The Antimicrobial Hydrogel has many benefits:

- Non-toxic and non-irritating
- Contains no steroids or antibiotics
- Contains no alcohol or iodine
- Supports reducing wound odor
- Adds moisture to dry wound beds
- Relieves itching and pain associated with dermal irritations
- Moistens and lubricates absorbent wound dressings
- Environmentally friendly—no special disposal required



INDICATIONS FOR USE:

Puracyn Plus Professional Formula Antimicrobial Hydrogel is an FDA-cleared 510(k) product for use by healthcare professionals to treat the following types of wounds:

- Acute and dermal wounds
- Partial and full-thickness wounds
- First and second-degree burns
- Diabetic ulcers and stasis ulcers
- Stage I-IV pressure ulcers
- Surgical wounds (incisions, donor and graft sites)
- Abrasions and skin irritations
- Trauma wounds
- Various dermatoses including atopic dermatitis

DIRECTIONS FOR USE:

- Cleanse the wound with Puracyn Plus Professional Formula Wound Irrigation Solution
- Shake Antimicrobial Hydrogel well before use
- Apply Antimicrobial Hydrogel
- With some wounds, a wound filler may be needed to fill the cavity
- Cover wound with appropriate wound dressing that holds Antimicrobial Hydrogel to the wound bed
- Change dressing as ordered or when needed

BILLING INFORMATION

Hydrogel dressings are covered when used on full-thickness wounds with minimal or no exudate (e.g. Stage III or IV ulcers). Antimicrobial Hydrogel dressings are not usually medically necessary for Stage II ulcers. Documentation must substantiate the medical necessity for use of Antimicrobial Hydrogel dressings for Stage II ulcers (e.g. location of ulcer is in the sacrococcygeal area). Usual dressing change for Antimicrobial Hydrogel wound covers without adhesive border or hydrogel wound fillers is up to once per day. The quantity of Antimicrobial Hydrogel filler used for each must not exceed the amount needed to line the surface of the wound. Additional amounts used to fill a cavity are not medically necessary. Documentation must substantiate the medical necessity for code A6248 billed in excess of 3 units (fluid ounces) per wound in 30 days. Use of more than one type of Antimicrobial Hydrogel dressing (filler, cover, or impregnated gauze) on the same wound at the same time is not medically necessary.

DISCLAIMER: This information does not guarantee reimbursement, but provides guidance for accurate documentation and appropriate usage for hydrogel wound filler. Should you need further assistance or specific coding questions, please contact your regional DMERC or intermediary. It is the manufacturers intent to share this information with healthcare professionals to highlight awareness of the reimbursement process.

Puracyn Plus Antimicrobial Hydrogel contains hypochlorous acid, a known antimicrobial preservative, which is known to inhibit the growth of microorganisms in the hydrogel.

For more information or to order Puracyn[®] Plus Professional Formula Antimicrobial Hydrogel, call customer service: 1.866.318.3116 (8am PST - 5pm PST)

Item Number: 6542 | HCPCS Code: A6248