PROVOX[®]

Adhesive Remover

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden.

For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents.

Ingredients/ Inhaltsstoffe/ Ingrediënten/ Contient/ Ingredienti/ Ingredientes/
Ingredientes/ Innehållsämnen/ Indholdsstoffer/ Innhold/ Ainesosat/
Innihaldsefni/ Koostis/ Sudétis/ Složení/ Összetevők/ Zloženie/ Sestavine/
Składniki/ Ingrediente/ Sastojci/ Sastojci/ Συστατικά/ Ингредиенты/
Съставки/ İçindekiler/ Βαղανηχηγυτη/ Bahan-bahan/ 成分/ 성분/ 成分/
: המכונות / מרכיבים

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Instructions for use

Intended Use

Provox Adhesive Remover is a single use wipe to help laryngectomized patients remove Provax Adhesives and Provax Silicone Glue.

CONTRAINDICATIONS

Hypersensitive or breached skin or known allergies to ingredients.

Application

Stop the additional oxygen supply, if applied, and remove the HME before applying Adhesive Remover.

When using a Provax FlexiDerm, Provax XtraBase, or Provax StabiliBase, first apply the Provax Adhesive Remover on top of the adhesive. This is not necessary for the other adhesives. Then, for all Provax adhesives, grasp the finger lift tab and apply Provax Adhesive Remover on the edge and underneath the adhesive. Carefully remove the adhesive and adhesive residue from the skin. Clean the skin with water and soap or use Provax Cleaning Towel afterwards.

PRECAUTIONS

- Avoid deep inhalation of solvent during application and be careful that liquid does not drip into stoma.
- In case you experience signs of irritation or redness, discontinue use of product. If the irritation persists, consult your physician.

WARNINGS

- For external use only;
- Keep out of reach of children;
- Avoid contact with eyes. In the case of accidental contact, flush eyes well with water;
- Do not apply directly to open wounds and mucous membranes;
- Single patient use only. Re-use may cause cross contamination.
- Do not cut the tissue in pieces, small pieces may be aspirated.

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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