




Derma Shield™ protective arm and leg sleeves are a non-compression skin covering that help provide a protective barrier against tears and abrasions for fragile skin tissue and adapts to different body shapes. Protective arm and leg sleeves can also help prevent patient disruption of medical treatments by camouflaging both IV sites and wound sites by blending in with the patient's skin tone.

**RECOMMENDED USE:** Patients with fragile skin; to protect IV / wound sites and skin from tears and abrasions.

### FITTING GUIDELINES - MEASURE BY THE BICEP AND THE CALF.

Protective Skin Sleeves are intended to fit comfortably. Actual fit will depend on individual measurements and patient comfort level. NYOrtho Protective Arm Sleeves will stretch up to 70% in size of both width and length. Use proper size, over stretching will result in a tighter fit adding to the amount of compression on the patient's skin and muscle.

### EASY TO IDENTIFY ARM SIZING

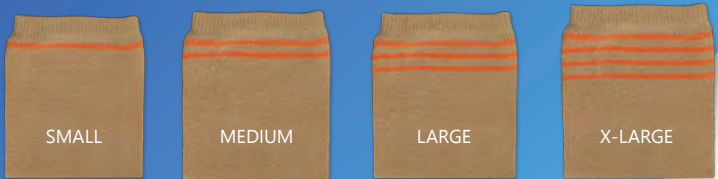

### INSPECTION

Inspect before each use: check for broken stitches; or torn, cut or frayed material; DO NOT use soiled or damaged products.

### ARM APPLICATION INSTRUCTIONS

1. Before applying the Derma Shield, roll the inside edge of the arm opening downwards until you reach the thumb opening.
2. Slip the arm opening over the patient's hand, placing the patient's thumb through the thumb hole.
3. Unroll the Derma Shield back up over the arm and adjust to a comfortable fit.
4. To uncover the palm, the patient's thumb may be removed from the thumb opening and the Derma Shield may be rolled up on the wrist.



# NYOrtho

## Derma Shield™



ANTIMICROBIAL



NOT MADE  
FROM NATURAL  
RUBBER LATEX



MACHINE  
WASHABLE



PROTECTS  
SKIN



4-WAY STRETCH  
SOFT MATERIAL

**TO LAUNDER** - Machine wash and dry at low temperature. For best results, hang dry.



### LEG APPLICATION INSTRUCTIONS

1. Before applying the Derma Shield, roll the inside edge of the leg opening downwards until you reach the heel curvature.
2. Slip the leg opening over the patient's foot until the heel curvature is positioned over the heel.
3. Unroll the Derma Shield back up over the leg and adjust to a comfortable fit.

### WARNING

Inspect skin frequently for infection or damage and determine appropriate treatment. Change product with new or laundered product in accordance with facility guidelines.

| Style | Model   | Size      | Circumference | Length | Unit |
|-------|---------|-----------|---------------|--------|------|
| Arm   | 9402-S  | Small     | 8½"           | 15½"   | Pair |
| Arm   | 9402-M  | Medium    | 11"           | 16½"   | Pair |
| Arm   | 9402-L  | Large     | 11"           | 18"    | Pair |
| Arm   | 9402-XL | XL        | 13"           | 20"    | Pair |
| Leg   | 9100-U  | Universal | 11"           | 13½"   | Pair |