

Door Guard

For wandering patients; to deter unauthorized entry or exits.



The **NYOrtho Door Guard** serves as an effective visual deterrent designed specifically to prevent wandering residents, such as those with Alzheimer’s or Dementia, from accessing restricted areas. Not only does it bolster fall management efforts, but it also actively dissuades confused individuals from entering zones that could be harmful or off-limits to them.

- Catches the eye and deters wandering.
- Reduces the risk of falls by preventing unauthorized access.
- Discouraging confused residents from entering off-limit zones.
- Attaches easily with adhesive hook and loop strips to the door frame and removes easily in case of emergency.
- Available in multiple sizes to suit a variety of doorways.
- Can be used with Door Guard Alarm (9797-ALRM).

The new and improved **NYOrtho Door Guard Alarm System** is a valuable tool designed to help prevent wandering and enhance safety for patients or residents in healthcare settings, as well as at home.



- New simplified setup
- Easy Installation
- Reliable alarm activation
- Battery Operated



Magnet attaches to alarm.
Done!

Door Guard	Dimensions	Door Guard W/Alarm
9797-40	12 x 40 in.	9797-40-ALRM
9797-48	12 x 48 in.	9797-48-ALRM
9797-53	12 x 53 in.	9797-53-ALRM
9797-60	12 x 60 in.	9797-60-ALRM

PROP65 WARNING: This product can expose you to chemicals including diethylhexyl phthalate or bisphenol A, which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to: www.P65warnings.ca.gov.

Door Guard Alarm

For wandering patients; to deter unauthorized entry or exits.

Alarm 9797-ALRM



Purpose: It alerts caregivers when a patient, resident, or even a child attempts to exit or enter a door monitored by the system, making it useful for fall management and wander prevention.

Simplified Setup: The updated design includes a metal trip plate on the alarm and a magnet sewn directly into the banner. To set up, simply attach the magnet to the plate, and the system is ready to function—no complicated steps required.

Easy Installation: Banner and alarm attach securely with adhesive hook and loop strips, and can be easily removed in an emergency.

Battery Operated: The system runs on a 9-Volt battery (included) and features a low battery indicator to ensure continuous operation.

This product is used in conjunction with **Door Guard STOP Sign (9797-XX)** banner to deter unauthorized entry or exit by unauthorized patients.

WARNING: CAREFULLY READ THE ENTIRE INSTRUCTION MANUAL PRIOR TO USE. FAILURE TO COMPLY WITH INSTRUCTIONS, WARNINGS AND CAUTIONS MAY RESULT IN SERIOUS INJURY TO PATIENT. THIS PRODUCT IS NOT A REPLACEMENT FOR ROUTINE VISUAL MONITORING. TEST ALARM DAILY.

Instructions

- 1. Connect Battery:** On the back of the alarm, at the bottom, lift open the battery compartment as shown (Fig. B). Connect the 9 volt battery. Close the door.
- 2. Location:** To determine the location of the alarm, first center the banner over the door and at the desired height, with the two ends of the banner on the wall. Use a little tape to hold the banner in place. There are two magnets sewn into the banner on both ends towards the top of the banner. Decide which side of the door you will place the alarm. With a pencil, mark the location of the magnet on the wall, that is sewn into the banner on the side you wish to place the alarm.
- 3. Secure Alarm to Wall:** Remove the banner from the wall. On the back of the alarm, peel the protective film off the adhesive hook/loop and firmly apply the alarm to the wall so that the magnet receptor will be forward facing over the circle drawn.
- 4. Placement of Stop Sign Banner:** Once the alarm is firmly attached to the wall, position and attach the banner magnet sewn at the top end of the banner over the alarm magnet sensor contact (Fig. A). Remove the protective film off the back of the loop provided with this kit and firmly apply to the wall where the banners hook will be placed. Do the same on the opposite side of the banner, apply the adhesive loop to the wall on the other side of the entrance and attach the banner to it.
- 5. Test the Alarm:** Detach the magnet from the base. If the alarm does not sound, check the battery.

Alarm Limited Warranty

The monitor is warranted by the Manufacturer to be free of defects in materials and workmanship for a period of one year from the date of purchase. If any warranty is breached, the customer shall immediately upon delivery notify their distributor of the specific nature to the defect. The Manufacturer will at its sole discretion, repair or replace defective components or products and return to the customer at customers expense. Customer is responsible for returning defective products to the Manufacturer including shipping costs, insurance and arranging carrier pickup. Products may only be returned with a valid Return Authorization Number. The Manufacturer is not responsible for warranty service for wear and tear due to normal use or product that is not properly cleaned, maintained, is abused, misused, dropped, or is damages by disasters such as but not limited to lightning or fire.

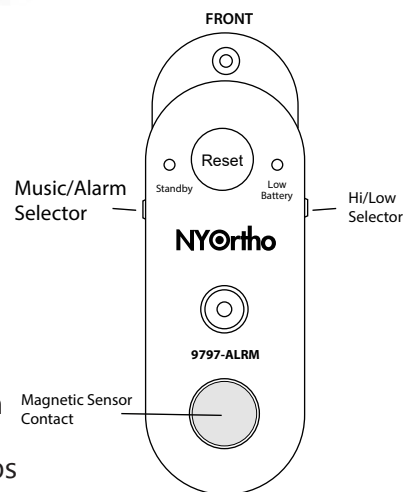


Fig. A

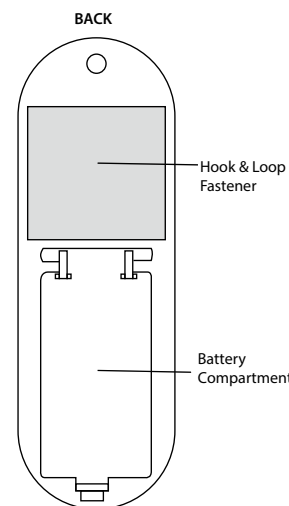


Fig. B

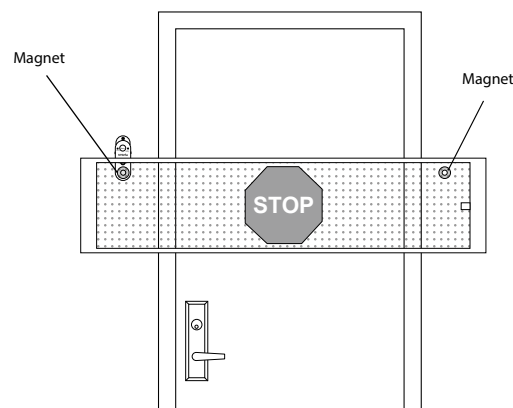


Fig. C