




OPTIONAL PLASTIC HALF RAIL INSTALLATION AND OPERATION

KIT ZA84200 - (ILLUSTRATION SHOWS RAIL POSITIONED ON RIGHT SIDE OF BED)



1. Position the Plastic Half Rail (#2) as shown, with the taller, straight edge toward the head end of the bed and arms angled toward the center of the bed.
2. From the outside, insert the two welded grooved pins on the rail arms into the two large holes in the side of the head deck.
3. From the inside, install the metal hair pin clip end of the Lanyard Assembly (#3) onto the exposed grooves at the end of the pins.
4. Position the other terminal end of the Lanyard Assemblies (#3) over the small hole on the inside of the deck frame. Secure with a small Phillips Screw (#4).

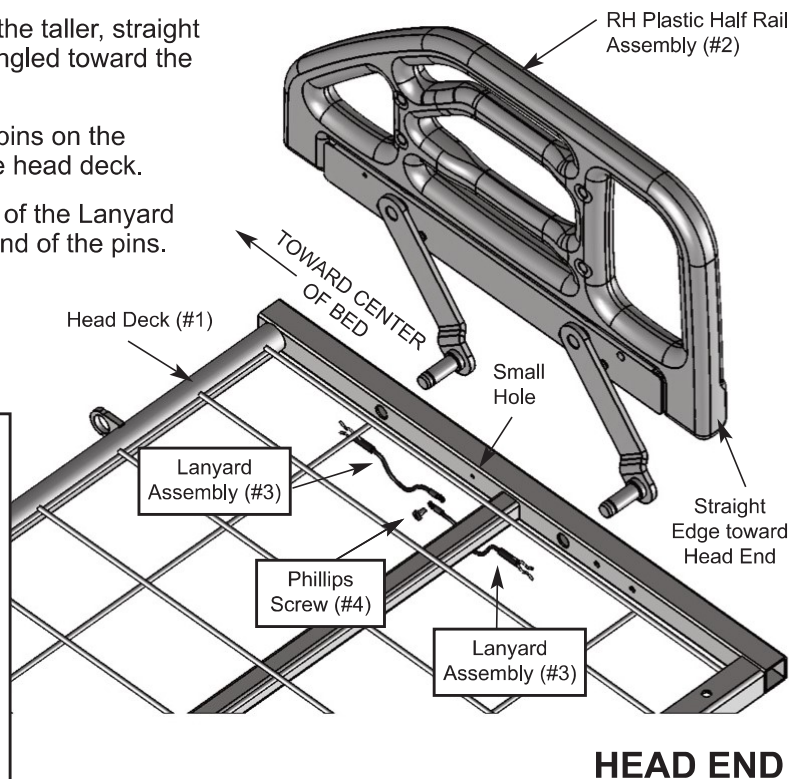

ORDER PLASTIC RAIL KIT #ZA84200, WHICH INCLUDES A RIGHT AND LEFT HALF RAIL, FOUR LANYARD ASSEMBLIES, AND TWO PHILLIPS SCREWS (OR TWO OPTIONAL HEX HEAD SCREWS). ORDER BAG #999-0842-905 FOR JUST THE REPLACEMENT HARDWARE.

THE PLASTIC RAIL CAN BE INSTALLED ON EITHER THE HEAD OR FOOT END OF 80" BEDS. **IT SHOULD ONLY BE INSTALLED ON THE HEAD END OF 76" BEDS.**

PLASTIC HALF RAIL OPERATION

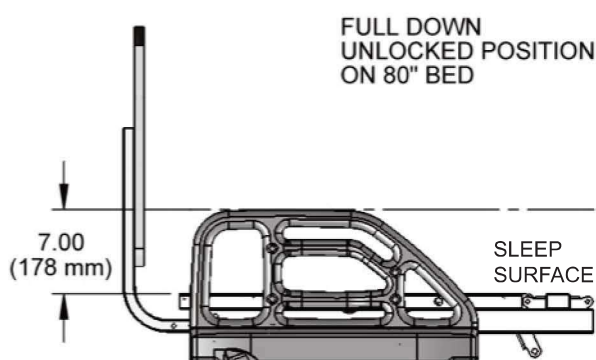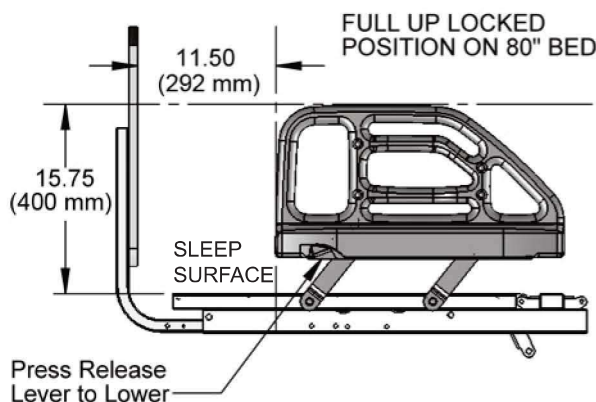

! TO AVOID PINCHING FINGERS, HOLD THE HALF RAIL FROM THE TOP WITH ONE HAND AND CAREFULLY DEPRESS THE BLACK RELEASE LEVER WITH THE OTHER. ENSURE THERE ARE NO OBJECTS IN THE WAY THAT MAY HINDER THE FULL MOVEMENT OF THE RAIL.

! TO AVOID PERSONAL INJURY, ENSURE THE HALF RAIL IS ALWAYS IN THE FULL UP LOCKED POSITION WHENEVER RESIDENT/PATIENT IS LEFT UNATTENDED.

! POWERED AIR MATTRESS SURFACES MAY POSE A RISK OF ENTRAPMENT. PRIOR TO USE, ENSURE THE THERAPEUTIC BENEFITS OUTWEIGH THE RISK OF ENTRAPMENT.