

ITEM# B3000



TOILET SAFETY FRAME

WEIGHT CAPACITY: 250 LBS




Rhythm Healthcare, LLC | Lifestyle Mobility Aids
3200 Tyrone Blvd N
St. Petersburg, FL 33710

contactus@rhythmhc.com
www.rhythmhc.com

GENERAL EXPLANATION

- ⚠ Read and understand manual prior to use. Save for reference. If components are damaged or missing, contact your distributor immediately. Do not substitute parts.

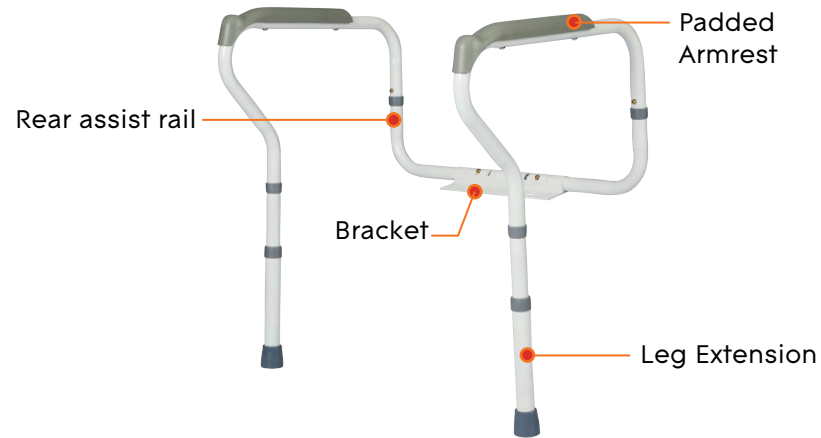

SAFETY PRECAUTIONS

- > Consult a health care professional for advice and instructions specific to your needs before use.
- > Before use, ensure push buttons are clicked into place securely.
- > Always use caution on wet or slippery surfaces.

MAINTENANCE

- > Wipe weekly with a soft cloth and disinfectant.
- > Do not use organic solvents, such as acetone or abrasive cleansers.
- > Inspect weekly for wear.
- > If any component appears worn or is not working properly, contact your dealer for repair and/or appropriate replacement parts.

ASSEMBLY INSTRUCTIONS


1. Detach seat from toilet bowl. Place bracket of rear assist rail on back of toilet bowl w/ two holes in bracket positioned over the holes used to fasten the toilet seat.

2. Place toilet seat on top of bracket, aligning all holes and fasten with toilet seat bolts.

3. Push in brass button and slide rear assist rail to appropriate hole of bracket for desired arm width, until button clicks into place.

4. Attach the armrests to the rear assist rail making sure brass buttons click into place.

5. Adjust armrest leg extensions according to the height of toilet and making sure the brass buttons click into place.

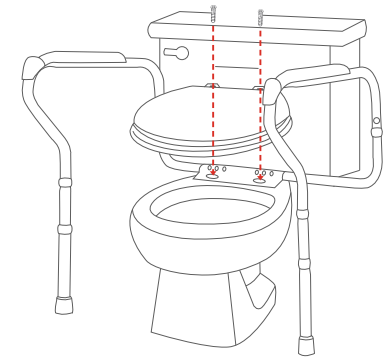

OPERATING INSTRUCTIONS

Stand with your back to center of seat, take hold of armrests and slowly sit down. Leg extensions should be adjusted so that your feet are flat on floor when seated.

WARRANTY

Limited Lifetime Warranty against manufacturing defects. This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications.

It is recommended but not required that the product be returned to the Rhythm Healthcare, LLC / Lifestyle Mobility Aids dealer through whom it was originally purchased. However all Rhythm, LMA dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Rhythm/LMA, prior authorization will be required. In the event of a defect covered by this warranty, Rhythm/LMA will determine if the product will be repaired or replaced.