



Shower Chair

AVAILABLE COLORS:



B3200

USER MANUAL - SHOWER CHAIR

GENERAL EXPLANATION OF YOUR SHOWER CHAIR



USER GUIDE

⚠ Prior to use please read all instructions.

We recommend you keep this User Guide in a safe place for future reference.

TABLE OF CONTENTS

SPECIFICATIONS	2	• Tool free assembly • Easysnap together assembly
SET UP	3	• Blow molded bench and back provides comfort and strength
CLEANING	3	• Drainage holes in bench reduce slipping • Convenient shower sprayer holder
WARRANTY	3	• Aluminum frame is lightweight, durable and corrosion proof • Angled legs with suction style tips provide additional stability

SPECIFICATIONS

NAME	Shower Chair
WEIGHT	5 lbs
SEAT HEIGHT:	16" to 21"
SEAT DIMENSIONS	20" W x 12" D
UNIT DIMENSIONS	18.5" x 17"
WEIGHT CAPACITY	300 lbs

SET UP

Chair Assembly

- 1 Slide the narrow end of each leg into one of the leg openings on the base, push and turn until brass push button is completely through the hole on the leg opening. This should make a "click" sound [A].
- 2 Adjust height using numbered holes and brass push button. Turn stool over and test for stability. If unstable check to make sure all the legs are at the same height (number) [B].
- 3 Turn the seat upright with the backrest tubing holes facing you. Insert the narrow end of the "L" shaped backrest tubes into the attached tubing until push button "clicks" into base. Insert the back into the opening of the "L" shaped tubes. Check buttons are completely engaged through the holes before using [C].



Adjust Leg Height

- 1 Adjust the leg height by pushing the button and sliding the tube up or down. The height will adjust one hole at a time.
- 2 Ensure that all legs are adjusted to the same height.

CLEANING

! Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.

- › Use a non-abrasive detergent or household cleaner with warm water.
- › Rinse with warm water and wipe dry.
- › DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

LIFETIME LIMITED WARRANTY

Your Lifestyle Mobility Aids product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

Should you have any questions regarding your Lifestyle Mobility Aids product or warranty please call your local dealer or contact us directly.



Some Additional Products:



Premium Shower
Chair with Back
and Padded Arms



Commando Memory
Forearm Crutch



Super Lite 19"
Aluminium
Companion Chair

Designed In The USA and Distributed By:
LIFESTYLE MOBILITY AIDS
Email : contactus@lifestylemobilityaids.com
Website : www.lifestylemobilityaids.com