



Folding Powder Coated Steel Commode

300 LBS
WEIGHT CAPACITY



B3400FEC

LIFESTYLE MOBILITY AIDS
Email : contactus@lifestylemobilityaids.com
Website : www.lifestylemobilityaids.com

GENERAL EXPLANATION




ASSEMBLY INSTRUCTIONS

⚠ If Commode Seat has been exposed to cold, allow to reach room temperature before assembly to prevent cracking.

⚠ Read entire manual prior to assembly and save for future use.

To set up Commode Frame

- 1 Unfold frame making sure sides of frame are at right angles to the rear of the frame and the open ends of Cross Brace Sockets are facing the center of the frame.
- 2 Swing the Bucket Support Frame up and insert the ends of the cross brace into the Cross Brace Sockets on each side frame by depressing the push pins and pushing until they click securely into place.
- 3 Insert the Back Brace into back of frame making sure push pins click securely into place.
- 4 Attach Commode Seat and Lid to rear of frame by pressing clips onto rear Cross Brace until they snap securely in place.

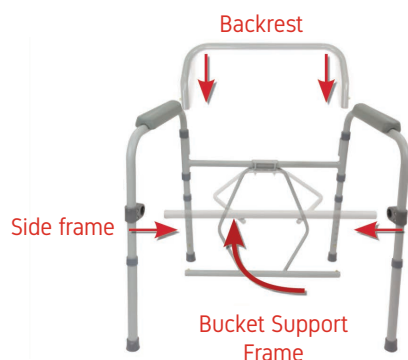

To adjust Legs

- 1 Depress the push pin and slide the Lower Leg up or down to desired height until push pin clicks securely into adjustment hole.
- 2 Repeat step 1 for each Leg making sure they are all set to the same height.

OPERATING INSTRUCTIONS

Using as Bedside Commode

- 1 Lift Commode Seat and place Commode Bucket on Support Rails making sure it is directly below the opening; then, lower Commode Seat for use.
- 2 Place the lid on the Commode Bucket when not in use to contain odor and use the handle on Commode Bucket to remove for cleaning.

Using Commode over Toilet

- 1 Adjust height, if necessary, and remove the Back Brace; then, replace the Commode Bucket with the Splash Guard.
- 2 Lift the toilet bowl seat and place the Commode over the toilet centering the Splash Guard directly over the toilet bowl; then, lower the Commode Seat for use.

SAFETY PRECAUTIONS

⚠ Lifestyle Mobility Aids, Inc. is not liable for personal injury or property damage due to failure to follow the instructions or recommendations in this manual or improper use of this product.

- › If you do not understand these instructions, contact your dealer or a health care professional for assistance.
- › If any parts are missing or damaged, contact your dealer for replacement. Do not substitute parts.
- › Ensure Commode Seat is securely attached to frame and all legs are set at the same height before use.
- › Users with limited physical strength will require assistance/supervision.
- › Weight capacity is 300 lbs (evenly distributed).

MAINTENANCE

- › Clean after each use with disinfectant.
- › DO NOT use organic solvents, such as acetone or abrasive cleansers.
- › Inspect weekly for wear and to ensure all components are secure. If any component appears worn or is not working properly, contact your dealer for repair and/or appropriate replacement parts.

WARRANTY

Limited Lifetime Warranty against manufacturing defects.

This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

This warranty does not extend to non-durable components which are subject to normal wear and tear and need periodic replacement.

For Warranty Service, it is recommended but not required that the product be returned to the Lifestyle Mobility Aids dealer through whom it was originally purchased. However, all Lifestyle Mobility Aids Dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Lifestyle Mobility Aids, prior authorization will be required. In the event of a defect covered by this warranty, Lifestyle Mobility Aids will determine if the product will be repaired or replaced.

Lifestyle Mobility Aids product are identified by a serial or lot number label at the time of manufacture. Removing or defacing the label may void this warranty. If you have questions about your Lifestyle Mobility Aids product or this warranty, please contact us at contactus@lifestylemobilityaids.com.