



300 Held Drive
Northampton, PA 18067
Phone: (610)-262-6090
Fax: (610)-262-6080
Website: www.precisionmedical.com

April 12, 2017

*Precision Medical, Inc. and Medical Fittings
300 Held Drive
Northampton, PA 18067*

Addendum To Precision Medical Flowmeters User Manual

To Whom It May Concern,

This letter is to supplement cleaning instructions for Precision Medical Air and Oxygen Flowmeters whether as single or with PTO to include flowmeters installed on wye blocks or tee branches. Precision Medical Air and Oxygen Flowmeters are not single patient/disposable items. This includes:

*1MFA series
3MFA series
4MFA series
6MFA series
7MFA series
8MFA series
19MFA series*

Flowmeters can be externally cleaned/disinfected using hospital approved wipes with quaternary cleaning, sodium hypochlorite, or hydrogen peroxide.

Alcohol based wipes should not be used to clean/disinfect oxygen flow meters and accessories due to the risk of fire.

We recommend following the directions of the manufacturer of the chosen cleaner.

The flowmeters should be wiped down between patient use.

Precision Medical, Inc.