

## INGREDIENTS

Water (Aqua), Propylene glycol, Cetareth-25, Polysorbate 80, Sodium benzoate, Ethylhexylglycerin, Propanediol, Citric acid, Argania spinosa kernel oil, Lactic acid, Tetrasodium glutamate diacetate, Aloe barbadensis leaf extract, Rhamnose, Glucose, Glucuronic acid

**CLOTHS:** Pulp/Polypropylene/Polyester Blend

**SIZE:** 7.9" x 7.9"

**Caution:** For external use only. Keep out of eyes. If contact occurs, rinse eyes thoroughly with water. If condition worsens or irritation persists, consult a doctor.

**REF** DFC100

4526-650

Supplied by:



**GOJO Industries**

PO Box 991  
Akron, OH 44309

©2015 GOJO Industries, Inc.  
Made in France.  
All Rights Reserved.

**Distributed by:**

**BARD** | MEDICAL

8195 Industrial Blvd  
Covington, GA 30014

## DIRECTIONS:

- Intended for post insertion daily/PRN Foley care.
- Non-sterile, single patient use.
- Perform clean technique per step. Always wipe away from insertion site.
- Place clean underpad beneath patient and remove all 5 wipes.
- Perform steps on front label taking care not to dislodge Foley catheter.
- Discard each wipe after single use per step.
- Remove orange compliance sticker from front of package and apply to BARD® Foley calendar located on BARD® urine meter or bag.

**DISCARD IMMEDIATELY IF  
DRY OR DAMAGED**

**DO NOT MICROWAVE**

**DO NOT FLUSH**

