Reusable Chamber

REF MYAIRVO**CHAMBER1**








Thank you for choosing F&P myAirvo™

Please read this entire user manual before you first use your reusable chamber. Following the steps in this manual, and referring to the images and diagrams will assist you with daily use of your F&P myAirvo device and its consumables.

LANGUAGES

PAGE	LANGUAGE	PAGE	LANGUAGE
1	English (en)	56	Română (ro)
4	Italiano (it)	59	Ελληνικά (el)
7	Deutsch (de)	62	Slovenský (sk)
10	Français (fr)	65	Česky (cs)
13	Español (es)	68	한국어 (ko)
16	ภาษาไทย (th)	71	Русский (ru)
19	Português brasileiro (pt-br)	74	简体中文 (zh)
22	Português (pt)	77	日本語 (ja)
25	Nederlands (nl)	80	Bahasa Indonesia (id)
28	Dansk (da)	84	Hrvatski (hr)
31	Suomi (fi)	87	Slovenščina (sl)
34	Norsk (no)	90	Magyar (hu)
37	Svenska (sv)	93	فارسی (fa)
40	Türkçe (tr)	96	български (bg)
43	العربية (ar)	99	Eesti keel (et)
46	עברית (he)	102	Latviešu valoda (lv)
49	繁體中文 (zht)	105	Lietuvių kalba (lt)
52	Polski (pl)		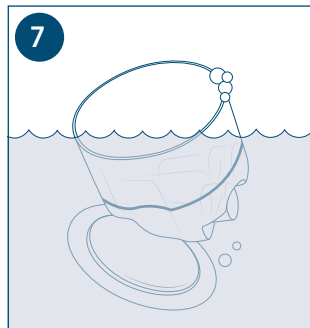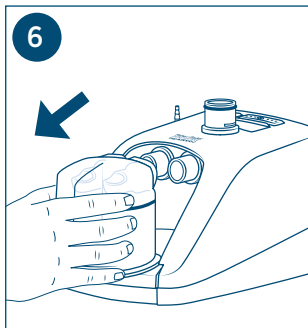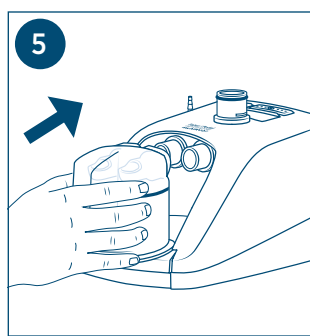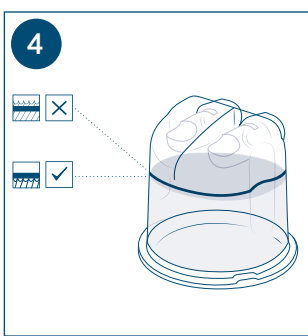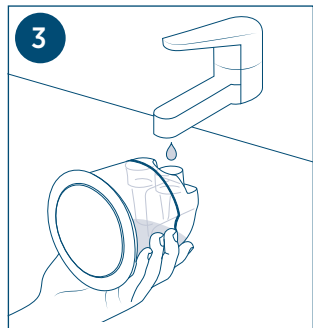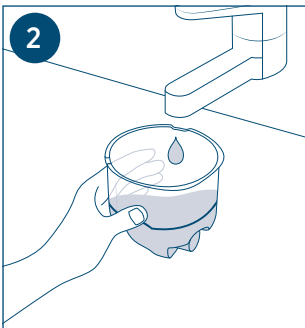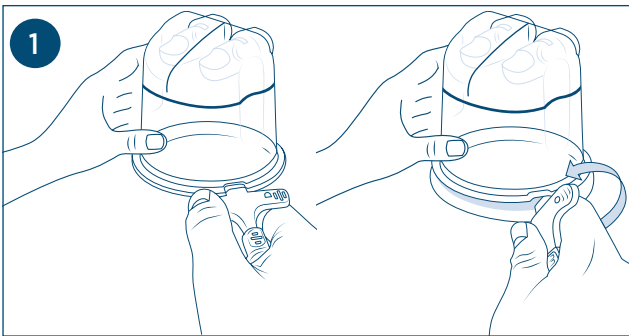

Rx only	Prescription only
	Single patient multiple use
CE 0123	Conforms with Regulation (EU) 2017/745
MD	Medical Device
	Contains Hazardous Substances

F&P and myAirvo are trademarks of Fisher & Paykel Healthcare Limited.
For patent information, see www.fphcare.com/ip

Manufacturer
Fisher & Paykel Healthcare Ltd,
15 Maurice Paykel Place,
East Tamaki, Auckland 2013
PO Box 14 348 Panmure,
Auckland 1741,
New Zealand
Tel: +64 9 574 0100
Fax: +64 9 574 0158
Email: info@fphcare.co.nz
Web: www.fphcare.com
Importer/ Distributor
Australia (Sponsor)
Fisher & Paykel Healthcare
Pty Ltd,
19-31 King Street, Nunawading,
Melbourne, Victoria 3131.
Tel: +61 3 9871 4900
Fax: +61 3 9871 4998
Austria
Tel: 0800 29 31 23
Fax: 0800 29 31 22
Benelux
Tel: +31 40 216 3555
Fax: +31 40 216 3554
Brazil
Fisher & Paykel do Brasil,
Rua Sampaio Viana, 277 cj 21,
Paraíso, 04004-000,
São Paulo – SP, Brazil
Tel: +55 11 2548 7002
China
代理人/售后服务机构:
费雪派克医疗保健（广州）
有限公司, 广州高新技术产业
开发区科学城科丰路31号G12
栋301号
电话: +86 20 32053486
传真: +86 20 32052132

Denmark
Tel: +45 70 26 37 70
Fax: +46 83 66 310
Finland
Tel: +358 9 251 66 123
Fax: +46 83 66 310
France
Fisher & Paykel
Healthcare SAS, 10 Av. du
Québec, Bât F5, BP 512,
Villebon-sur-Yvette, 91946
Courtaboeuf Cedex, France
Tel: +33 1 6446 5201
Fax: +33 1 6446 5221
Email: c.s@fphcare.fr
Germany
Fisher & Paykel Healthcare
GmbH, Deutschland,
Österreich, Schweiz,
Wiesenstrasse 49,
D 73614 Schorndorf, Germany
Tel: +49 7181 98599 0
Fax: +49 7181 98599 66
Hong Kong
Tel: +852 2116 0032
Fax: +852 2116 0085
India
Tel: +91 80 2309 6400
Ireland
Tel: 1800 409 011
Fax: +44 1628 626 146
Italy
Tel: +39 06 7839 2939
Fax: +39 06 7814 7709
Japan
Tel: +81 3 5117 7110
Fax: +81 3 5117 7115
Korea
Tel: +82 2 6205 6900
Fax: +82 2 6309 6901
Mexico
Tel: +52 55 9130 1626

Norway
Tel: +47 21 60 13 53
Fax: +47 22 99 60 10
Poland
Tel: 00800 49 119 77
Fax: 00800 49 119 78
Russia
Tel. and Fax: +7 495 782 21 50
Spain
Tel: +34 902 013 346
Fax: +34 902 013 379
Sweden
Tel: +46 8 564 76 680
Fax: +46 8 36 63 10
Switzerland
Tel: 0800 83 47 63
Fax: 0800 83 47 54
Taiwan
Tel: +886 2 8751 1739
Fax: +886 2 8751 5625
Turkey
İthalatçı Firma: Fisher Paykel
Sağlık Ürünleri Ticaret Limited
Şirketi,
İletişim Bilgileri: Ostim
Mahallesi 1249,
Caddesi No:6, Yenimahalle,
Ankara, Türkiye 06374,
Tel: +90 312 354 34 12
Fax: +90 312 354 31 01
UK
Fisher & Paykel Healthcare Ltd,
Unit 16, Cordwallis Park,
Clivemont Road, Maidenhead,
Berkshire SL6 7BU, UK
Tel: 0800 132 189
Fax: +44 1628 626 146
USA/Canada
Tel: 1800 446 3908
or +1 949 453 4000
Fax: +1 949 453 4001

myAirvo Reusable Chamber

 MYAIRVOCHAMBER1

INTENDED USE / INDICATIONS FOR USE

The MYAIRVOCHAMBER1 is a reusable humidification chamber that stores and allows heating of water for the humidification of respiratory gases. For use with the myAirvo Series humidifiers.

SYSTEM SPECIFICATIONS

- Maximum volume: 560ml
- The chamber will last 7 hours when full and set to 25 L/min and 37°C.
- Permissible transport and storage conditions: -10°C to 50°C
- Refer to myAirvo Series Humidifier User Manual for permissible operating conditions.

SETUP

- myAirvo Series Humidifier (“myAirvo”). Refer to the myAirvo Series Humidifier User Manual for information on the myAirvo.

HOME AND LONG-TERM CARE FACILITY USE

The product is intended to be used for a maximum of 2 years for use with your myAirvo, provided that the myAirvo and MYAIRVOCHAMBER1 cleaning and maintenance instructions are followed.

SETUP INSTRUCTIONS

Step 1

Separate the chamber from the base with the removal tool. To separate the chamber from the base, insert the end of the chamber removal tool into the chamber slot and perform a twisting motion.

Step 2

Rinse the chamber in warm water before use.

Step 3

Fill the chamber with water.

Step 4

Place on a flat surface to check that the water level is no higher than the 560ml volume indicated.

Step 5

Fit the chamber to the unit by pressing down the finger guard and

sliding the chamber on, carefully aligning the holes at the top of the chamber with myAirvo ports. Push the chamber on firmly until the finger guard clicks into place.

Contact your Fisher and Paykel Healthcare representative for assistance with setup and use, or to report any unexpected operation.

Warnings:

- Do not fill past the maximum water level line to avoid choking.
- Do not fill the water chamber with hot water to avoid airway burns.
- Use of a non-approved accessory could impair performance or compromise safety.
- Notice to User: If a serious incident has occurred while using this device please inform your local Fisher & Paykel Healthcare representative and Competent Authority in your country.

Caution:

- For inhalation, use only drinkable quality water or distilled water with the reusable chamber.

CLEANING INSTRUCTIONS

DAILY

Step 6

After drying mode is completed, hold the finger guard down and slide the chamber out of the myAirvo device. Discard any remaining water.

Step 7

Separate the chamber from the base with the chamber removal tool. Refer to Step 1 to use removal tool.

Hand-wash the chamber parts in a tub of warm water with a mild dishwashing detergent. Ensure the chamber is visually clean.

Rinse the chamber thoroughly in fresh water. Ensure that all soap residue has been removed.

Leave the chamber parts to dry out of direct sunlight or heat before reassembling. Once dry, refit the base.

DISHWASHING

Once every 7 days, the chamber can be cleaned in a domestic dishwasher. Remove the base from the chamber. Place the chamber top and chamber bottom on the top shelf of the dishwasher and ensure it is placed so that it is able to drain.

REMOVAL OF MINERAL DEPOSITS

Mineral deposits in tap water may cause the chamber base to discolour over time. Should mineral deposits occur, you can reduce these by performing the below step.

Soak the chamber base in a solution of vinegar (1 part) to water (2 parts) for 10 minutes. Clean the chamber by following daily cleaning steps prior to next use.

Cautions:

- Discard the chamber if the maximum water level line is no longer legible.
- For single patient use only. Reuse between multiple patients may result in transmission of infectious substances. Attempting to reprocess will result in degradation of materials and render the product defective.

Note:

- Ensure the rubber O-ring around the base is not dislodged or damaged.

DISPOSAL INFORMATION

Dispose of product and packaging according to local guidelines. Long-term care facilities should discard according to their standard method for disposing of contaminated product.

SYMBOL DEFINITIONS

Catalogue number

Prescription only

Consult instructions for use

Caution

Authorised European Representative

Batch code

Date of manufacture

Shelf life expiry date

Single patient multiple use

CE 0123

Conforms with Regulation (EU)
2017/745

Storage temperature

Manufacturer

Discard as regular waste

Correct water level in the chamber

Incorrect water level in the
chamber

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