

KNEE WALKER

Weight Capacity: 300 lbs



PARTS DIAGRAM



ASSEMBLY INSTRUCTIONS

- 1 Disconnect the straps that hold the tiller/handle bar down.
- 2 Move the handle towards upright position until it snaps into place.




3 Move camlock across bar until it locks into place.



2

ADJUSTING KNEE PAD HEIGHT

Release the camlock then remove the locking pin.

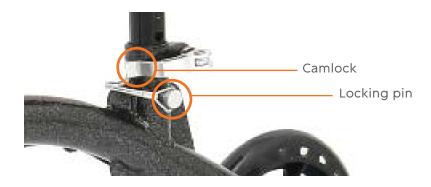

2 Adjust the knee pad for user.



3 Insert locking pin back into place and tighten the knob, making sure it is securely in place.

ENGAGING HAND BRAKE

1 To engage the brakes, squeeze and hold the brake levers on both handles.



1 To lock the brake, follow instructions above. Push and hold the button next to the brake lever. If done properly, the button will remain pushed in and the lever will stay locked. Rear wheels will stay in position when locked.

2 To release the brake, squeeze lever and the push button will release.

ADJUSTING HANDLE HEIGHT

1 To adjust the handle height, loosen the knob by turning it to the left until it can be removed.

2) Adjust the hand grip to the desired height and reinstall the adjustment knob by turning it clockwise until it is tightened.



OPERATING INSTRUCTIONS

• With brake applied, placement of injured leg on knee pad should be centered side to side and positioned forward to cover the full length of the knee pad.

2 With injured leg on knee pad, stand as straight as possible. (Adjust the height of knee walker pad as necessary.)

3 With injured foot pointing down, the propulsion leg should be maintained as close as possible to the knee pad.

4 Start out with small steps with propulsion leg while concentrating on keeping that leg next to the knee pad. With practice, you should be more comfortable with moving forward without deviating to the opposite side.

WARNINGS

- Before using the knee walker, you should be trained by a healthcare professional or dealer.
- Engage the lock and practice bending, reaching and transfers on and off the knee walker
- The use of the knee walker is individual to the person. You should develop your own methods for use based on your personal level of function and ability.
- Never attempt a maneuver that has not been practiced.
- Be aware of your surroundings when operating the knee walker. Look for hazards and avoid them.
- Avoid electrical cords, toys, and debris that could be a trip hazard.
- Do not use the knee walker on or near stairs or escalators.
- Use caution when moving from a carpeted area to hard surfaces or when changing levels, such as from a sidewalk to a street.
- Do not use while engaging in the use of mind-altering drugs or while suffering from dizziness.
- Use caution on uneven terrain, do not use in sand.
- Weight cap is 300 lbs.

MAINTENANCE

- > Clean frame with a damp cloth and dust weekly.
- > Clean upholstery and armrests with mild soap and water.
- > Never use abrasive cleaners or scouring pads.
- > If exposed to liquid, dry the knee walker thoroughly as soon as possible.
- > If any component appears worn or is not working properly, contact your dealer for repair and/or appropriate replacement parts.

WARRANTY

Limited Lifetime warranty against manufacturing defects. This warranty does not cover product failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications. Normal wear and tear on non-durable components, such as rubber accessories and casters are not covered under this warranty. These items are subject to normal wear and need periodic replacement.

For Warranty Service, it is recommended but not required that the product be returned to the Rhythm Healthcare, LLC dealer through whom it was originally purchased. However, all Rhythm Healthcare, LLC dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Lifestyle Mobility Aids, prior authorization will be required. In the event of a defect covered by this warranty, Rhythm Healthcare, LLC will determine if the product will be repaired or replaced.

This Rhythm Healthcare product is identified by a serial number at the time of manufacture. Removing or defacing the serial number label may void this warranty. If you have questions about your Rhythm Healthcare, LLC product or this warranty, please email us at contactus@rhythmhc.com.

Rhythm Healthcare, LLC 3200 Tyrone Blvd. N. St. Petersburg, FL 33710

contactus@rhythmhc.com (877) 843-6464