



19" Steel Companion Chair

ITEM #L1019



300 LBS
Weight Cap

OPERATING INSTRUCTIONS

TO OPEN CHAIR

Open wheelchair by tilting to one side and pushing down on seat rail until fully opened.

WHEEL LOCKS

Engage both wheel locks before getting in or out of chair, leaning forward in chair, or while in an elevator or wheelchair lift. Proper adjustment of wheel lock is at least 1/8" when locked. Locks are engaged by pushing handle completely forward. Adjustments to locks are made by loosening or tightening nut on the carriage bolt. Embed lock shoe at least 1/8" by sliding clamp toward rear wheel while handle is engaged in locked position. Tighten nut and bolt to secure in position. Test for correct locking action before actual use.

ADJUSTING LENGTH OF FOOT REST/LEG REST

When adjusting length of foot rests or leg rests, user should be sitting in wheelchair with feet on the foot plates. This product has push button height adjustable legs. Front rigging should not be so low that it causes uncomfortable pressure to thighs at front of seat upholstery. Footrest should not be lower than 2-1/2" from the ground to ensure clearance.



REMOVAL OF SWING-AWAY FOOT RESTS/LEG RESTS

Release lock by pushing on the front rigging lever. Swing away foot rest/leg rest out and sideways: Lift to remove.

TO CLOSE/FOLD CHAIR

First fold foot plates to vertical position. Grab seat with both hands at front and back edge and lift. Or, tilt chair to one side and lift up on seat rail.



MAINTENANCE

Do not use an abrasive cleanser that will scratch the finish.

Wipe chrome parts with a clean, soft cloth at least once a week. All metal parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately.

TIRES

Solid rubber tires need minimal maintenance, but should be cleaned occasionally with a damp cloth. Replace if tires become severely worn or cracking appears.

FRONT CASTERS

Casters should be checked periodically and lubricated occasionally by an authorized dealer.



WARNINGS

- Do not attempt to adjust or operate wheelchair without carefully reading all instructions in this manual.
- This wheelchair has not been approved as a seat that can be used within vehicles of any kind. Always transfer wheelchair user to a vehicle manufacturers approved seat and use safety restraints available.
- Do not operate this wheelchair on roadways, streets or any surfaces with vehicular traffic.
- Do not attempt to operate on inclines 10% or greater without proper assistance.
- Be sure to completely engage wheel locks when getting in and out of wheelchair or when in elevator or wheelchair lift.
- Wheelchair should be on a smooth, stable and level area with wheel locks engaged before transferring user in or out of wheelchair.
- When getting in or out of wheelchair, do not stand on or apply weight to foot rests.
- Do not lean over or apply weight to the top of wheelchair back. This can cause wheelchair to tip over. When trying to extend arms over back, only reach as far as your arm can extend without having to alter your seating position.
- Do not use wheel locks to slow your descent down an incline.
- Do not stand on wheelchair.
- Weight cap is 300 lbs.



LIMITED LIFETIME WARRANTY

Your Lifestyle Mobility Aids product is warranted to be free of defects in workmanship and materials for the lifetime of the original consumer.

This device was built to exacting standards and was carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will at our option, repair or replace the device.

This warranty does not cover device failure due to misuse or negligence, or normal wear and tear. This warranty does not extend to non-durable components, such as rubber accessories, castors or grips, which are subject to normal wear and tear and need periodic replacement.

If you have any questions about your Lifestyle Mobility Aids device, please contact your authorized Lifestyle Mobility Aids dealer.



www.lifestylmobilityaids.com

