Cleaning Instructions for Expression Collection Combo Kit and Manual Diaphragm Kit

PRIOR TO FIRST USE:

DO NOT boil tubing or tubing adapter

- Disassemble and boil all the parts, in the Kits, in water for 5 minutes.

- Be sure to use enough water to keep all parts afloat while boiling.
- Remove all parts from the water immediately after sterilizing.
- To avoid scalds and burns, **DO NOT** touch the sterilized parts with bare hands.

- Ensure all parts are completely dry before use. Partially dry components may affect the performance of your electric breast pump.

GENERAL CLEANING:

Set Aside Tubing, Tubing Connector and diaphragm Cap.

- These items do not need to be washed
- If any liquid or liquid vapor is found inside the tubing, follow the Tubing Cleaning Instructions carefully before use.
- All parts of the Kits, or Sweet Assist Handle may be cleaned in the upper rack of your dishwasher or hand-washed with dish soap in hot water.
- **DO NOT** use solvents or abrasives. Rinse with hot, clean water.
- To clean the main electric pump unit, wipe it with a wet wipe or damp cloth, using only water or mixed with milk soap.

TUBE CLEANING INSTRUCTIONS:

DO NOT Insert any objects into the tubing

- If liquids or liquid vapors are found inside the tubing, rinse the tubing with clean hot water.
- Hang to air-dry completely before using.

BEFORE EACH USE:

- Wash your hands thoroughly with soap and warm water before handling Kit parts, the electric breast pump, the manual pump kit, or any attachments.
- Make sure all parts are clean and dry.
- Ensure that the tubing is free from any liquid or liquid vapor.
- **DO NOT** operate your Rumble Tuff breast pump if any liquid or liquid vapor is visible in the tubing.

Rumble Tuff

865 N 1430 W Orem, UT 84057
801-609-8168 or 1-855-228-8388
customerservice@rumbletuff.com

DELICATE VALVES:

CAUTION: Be gentle when cleaning the Membrane Valve (One-Way Valve). If it gets damaged, your breast pump will not function properly. To clean the Membrane Valve (One-Way Valve), rub it lightly between your fingers in warm soapy water, then rinse thoroughly in clean water, allow to air dry. **DO NOT** insert objects into the Membrane Valve (One-Way Valve), this may cause damage to the valve.

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