



ASSURE® PLATINUM METER DISPLAY SCREEN

Full Display Screen Layout

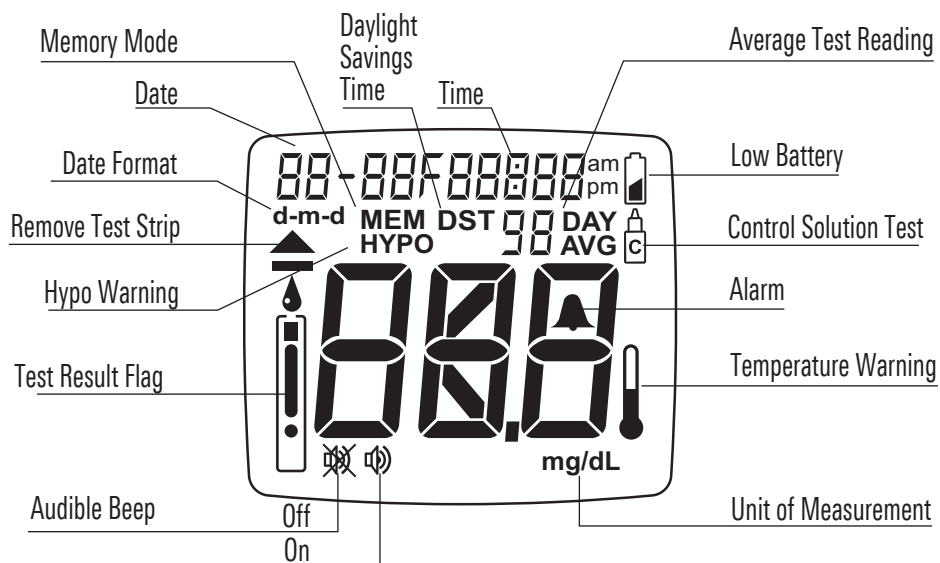

ASSURE® PLATINUM TEST STRIPS

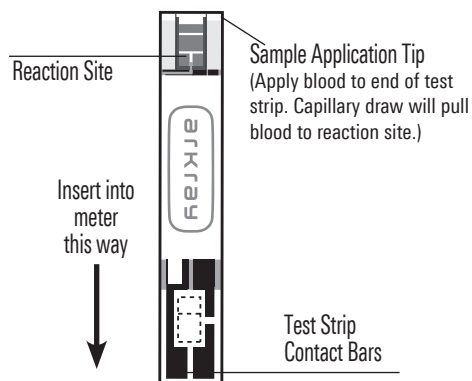

ORDERING DETAILS

	ARKRAY Product Code#	NDC#
Assure Platinum Test Strips 50-count	500050	08317-5000-50
Assure Platinum Test Strips 100-count	500100	08317-5001-00

The Assure® Platinum Test Strips come in a moisture-proof, light-protected bottle, and are designed to work with the Assure® Platinum meter. The test strips are sensitive to moisture and light, so it is important that you keep the bottle sealed until you are ready to use the strips. Do not leave any test strips outside the test strip bottle while not in use. Replace the cap tightly after a test strip is removed from the bottle.

These test strips require only 0.5 µL of blood. Apply the application tip to the blood sample. The strip draws in the blood by capillary action. The blood glucose reacts with the enzyme on the test strip. This produces a current proportional to the blood glucose level. The meter detects the current and converts it into a blood glucose reading. Assure® Platinum test strips use glucose oxidase chemistry.