

COVRSITE[◊]

Secondary Composite Dressing



Description

COVRSITE Cover Dressing is designed as a secondary cover dressing for gels (i.e., SOLOSITE[◊]), gel sheets, alginates, wound fillers and non-adhesive foams.

Benefits

- Can also be used as an alternative to tape and gauze over central line dressings.
- The gentle adhesive is designed so that the dressing can be changed daily.
- COVRSITE replaces traditional tape and gauze with a single step dressing.
- It is water-resistant, extensible, and conformable.

Indications

COVRSITE Cover Dressing is indicated for the management of acute or chronic wounds.

Ordering Codes / Reimbursement Information

Usual Allowable Amount: Up to 1 per day (gauze with border).

Specific Coverage Criteria: Covered when used on wounds with light to moderate exudate.

Product #	Size	Pcs/Pkg	Pkgs/Case	HCPCS Code
59714000	4 x 4	10	10	A6219
59714100	4 x 4	30	10	A6219
59714300	6 x 6	10	10	A6219
59714400	6 x 6	30	10	A6219

Instructions for Use

Step 1

Choose a dressing with a pad size slightly larger than the area of the wound.

Step 2

Remove the first tab from the underside of the dressing.

Step 3

Adhere the exposed adhesive side on one side of the wound.

Step 4

Remove the second tab and adhere the rest of the dressing over the wound.