

DERMAL WOUND CLEANSER

First Aid Antiseptic Spray Cleanser



Description

DERMAL WOUND SKIN/WOUND CLEANSER is an over-the-counter, non-toxic, non-irritating, no-rinse, first-aid antiseptic product.

Features & Benefits

- **Effective First-Aid Antiseptic**
Dermal wound contains the active antimicrobial ingredient Benzethonium Chloride to help prevent bacterial contamination in minor cuts, scrapes and burns.
- **Gentle to Skin**
Dermal wound contains Polysorbate 20, a gentle yet effective surfactant that makes cleansing of the skin and wound quick and easy.
- **Easy to Use**
Dermal wound is available in 8 oz. and 16 oz. adjustable nozzle trigger-spray bottles that allow the caregiver to gently dislodge wound debris and slough with a direct stream or irrigate the wound with the use of a fine mist spray.

Indications

DERMAL WOUND SKIN/WOUND CLEANSER is indicated as a first-aid antiseptic to help reduce the risk of infection in minor cuts, scrapes and burns. It is ideal for washing small superficial wounds and aids in the removal of exudate and other foreign material such as dirt and debris from skin and wounds.

Warnings

For external use only. Do not use in the eyes. In case of deep or puncture wounds, animal bites or serious burns, consult a doctor. Stop use and consult doctor if the condition persists or gets worse. Do not use longer than one week unless directed by a

doctor. Keep out of the reach of children. If swallowed, get medical help or contact a Poison Control Center immediately.

Ordering Codes / Reimbursement Information

Specific Coverage Policy: Supplies used in the management of incontinence, including creams, salves, lotions, barriers (liquid, spray, wipes, powder, paste) or other skin care products are non-covered by the DMERC.

Product #	Description	Bottles/Case	HCPCS Code
449000	16 oz. Spray Bottle	12	A6260
59449200	8 oz. Spray Bottle	12	A6260

Instructions for Use

Step 1

Clean the affected area.

Step 2

Apply a small amount of this product on the area 1 to 3 times daily.

Step 3

Rinse with saline or normal protocol.

Step 4

Cover wound with sterile dressing or bandage as needed.

Step 5

If applying dressing or bandage let dry first.