

Needlestick prevention that meets your needs



- ▶ virtually eliminates exposure
- ▶ single-handed, passive activation
- ▶ hands remain behind the sharp
- ▶ integrated safety mechanism
- ▶ prevents recapping and needle removal
- ▶ remains safe through disposal
- ▶ easy to use, requires minimal training
- ▶ safe and effective for patients

Catalog
Number Description

VanishPoint® U-100 insulin syringes	100 per box/800 per case
15291	0.5mL VanishPoint insulin syringe 30G x 3/16" NEW
15271	0.5mL VanishPoint insulin syringe 30G x 5/16"
15221	0.5mL VanishPoint insulin syringe 30G x 1/2"
10291	1mL VanishPoint insulin syringe 30G x 3/16" NEW
10271	1mL VanishPoint insulin syringe 30G x 5/16"
10281	1mL VanishPoint insulin syringe 29G x 5/16"
10211	1mL VanishPoint insulin syringe 29G x 1/2"

1mL VanishPoint® syringes	100 per box/800 per case
10131	1mL VanishPoint tuberculin syringe 27G x 1/2"
10151	1mL VanishPoint tuberculin syringe 25G x 5/8"
10161	1mL VanishPoint syringe 25G x 1"

VanishPoint® Allergy Syringe Tray	25 per tray/1000 per case
10134	1mL VanishPoint allergy syringes 27G x 1/2"

3mL VanishPoint® syringes	100 per box/600 per case
13021	3mL VanishPoint syringe 27G x 1 1/2"
10301	3mL VanishPoint syringe 25G x 5/8"
10391	3mL VanishPoint syringe 25G x 1"
13011	3mL VanishPoint syringe 25G x 1 1/2"
10311	3mL VanishPoint syringe 23G x 1"
10321	3mL VanishPoint syringe 23G x 1 1/2"
10331	3mL VanishPoint syringe 22G x 1"
10341	3mL VanishPoint syringe 22G x 1 1/2"
10351	3mL VanishPoint syringe 21G x 1"
10361	3mL VanishPoint syringe 21G x 1 1/2"
10371	3mL VanishPoint syringe 20G x 1"
10381	3mL VanishPoint syringe 20G x 1 1/2"

5mL VanishPoint® syringes	100 per box/600 per case
----------------------------------	--------------------------

10mL VanishPoint® syringes	100 per box/600 per case
-----------------------------------	--------------------------

5mL and 10mL VanishPoint® syringes are available with a variety of needle lengths and gauges. For a complete list, please contact RTI or visit our website at www.retractable.com.



**RETRACTABLE
TECHNOLOGIES, INC.**
www.retractable.com

CE
0086

© 2008, 2016 Retractable Technologies, Inc.
All rights reserved. M005 REV.14 EN, MAY 2016

511 Lobo Lane
Little Elm, Texas 75068-5295 USA
Tel.: (972) 294-1770
Fax: (972) 294-4400
Toll Free: 1-888-703-1010
rtisales@vanishpoint.com

UNCOMPROMISING SAFETY FOR CLINICIANS AND PATIENTS



VANISHpoint®
SYRINGES

The New Standard for Safety®

VANISHPOINT®
Safety by design

A study comparing the effectiveness of *passive* SED (safety-engineered devices) with that of *active* SED, concluded that passive SED (such as the VanishPoint® syringe) significantly reduced the needlestick injury rate.¹

Attached needle design:

- maintains a closed system
- prevents leakage and contamination
- reduces deadspace
- prevents unnecessary needle changes

"Changing needles between drawing vaccine from a vial and injecting it into a recipient is unnecessary."²

Safe for you and your patients



¹A. J. Goris, et al., "Reducing Needlestick Injuries from Active Safety Devices: A Passive Safety-Engineered Device Trial," *AJHP Journal*, vol. 34, no. 2 (Spring 2014), pp. 14-18.

²CDC (2002). General recommendations on immunization: recommendations of the Advisory Committee on Immunization Practices (ACIP) and the American Academy of Family Physicians (AAFP). *MMWR* 51(RR02):1-36.

1

Use standard procedure for drawing up medication.



2

Prepare and give injection using aseptic technique according to institutional policy.



3

While the needle is still in the patient, fully depress the plunger to activate retraction.



4

Dispose of VanishPoint® syringe in an appropriate sharps container.



the VANISHPOINT®

ADVANTAGE


Effective safety with automated retraction technology

Clear, unobstructed calibrations allow for accurate dosing

Triple beveled, lubricated needle provides patient comfort

EASY, ONE-HANDED ACTIVATION

NO CONTAMINATED SHARP EXPOSED

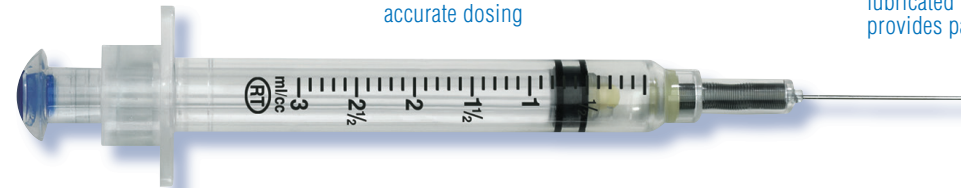

Pre-removal activation prevents exposure to contaminated sharp

Needle retracts directly from patient into syringe




VanishPoint® insulin syringes
NOW AVAILABLE WITH
3/16" (5mm) needles, and
5/16" (8mm) needles




Activated VanishPoint® syringes require less disposal space than other safety syringes and prevent disposal-related injuries.

Other Available Safety Products

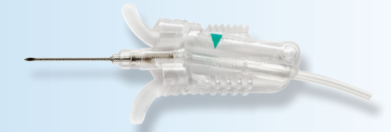

VanishPoint® Blood Collection Set

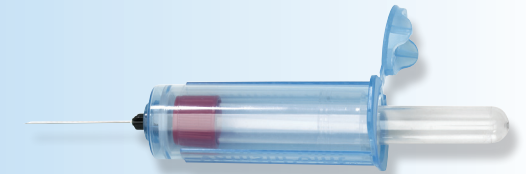

VanishPoint® Blood Collection Tube Holder



VanishPoint® IV Catheter



Patient Safe® Syringe